



TECHNICAL NOTE

INTRAORAL CLEANING OUTER SURFACES INTRAORAL DEVICES

N.2022-01

12/2022



1. CONCERNED PRODUCTS

- *X-MIND AC*
- *X-MIND DC*
- *X-MIND unity*
- *xgenus ac*
- *xgenus dc*

2. DESCRIPTION

In order to properly carry out the cleaning procedures of the intraoral devices, in the following points it is described how to clean the outer surfaces avoiding problems of corrosion and/ or removal of paint

3. CLEANING PROCEDURE



IMPORTANT



**TURN OFF AND DISCONNECT THE DEVICE FROM THE SUPPLY MAINS
BEFORE CARRYING OUT CLEANING OPERATIONS**

- **DO NOT USE A STEEL SPONGE, BUT ALWAYS SOFT CLOTHS OR COTTON WOOL**



- **For the spacer cone, always use a cotton wool**



- **DO NOT APPLY ANY OIL-BASED DETERGENT OR AGGRESSIVE DETERGENT**

- For extraordinary cleaning use detergents that **DO NOT** contain *alkaline solutions, saline solutions, amides, ketones, aromatic hydrocarbons, hexane, trichloroethane, acrylonitrile or dichloromethylene*
- **ABSOLUTELY USE ZERO CORROSION CLEANERS**
- Do not spray products directly on the device. Apply the product on a clean cloth



- Always use disposable protective covers for the applied parts
- Do not use UV systems to disinfect the equipment, as exposed parts of the device can turn yellow or discolour



- **DO NOT USE ALCOHOL OR ALCOHOL-BASED DETERGENTS ON THE LABELS OR CONES**
These latters are made of polycarbonate, and alcohol would brittle them



- To avoid any potential hazard or danger to operators and patients, contact your authorized Acteon Technical
- Representative immediately if you experience any unusual operation, mechanical issues, or equipment malfunction

For further information:
IMAGING AFTER SALES SUPPORT TEAM
imaging.italysupport@acteongroup.com
de Gotzen s.r.l via Roma 45
20157 Olgiate Olona (VA) - Italy